**EN\_**IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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EN

**Dear customer,**

Thank you for purchasing this product. So that your appliance serves you well, please read all the instructions in this user's manual. If you have any questions, please contact our customer care center, Our contact details are below:

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**EN****Maintenance**

1. The exterior of this rollator walker should be cleaned periodically with a soft damp cloth. Do not use abrasive cleaners or caustic chemicals.
2. Check the parts for any looseness, wear, or damage periodically. If any problems are found, correct them before using them.
3. If haven't used it for a long time, please store it in a cool, dry and safe place.

**Warning**

1. Before using, make sure the rollator is fully opened.
2. Checking the brakes and seat are locked, seat part especially.
3. The brakes must be locked before using the seat.
4. Max loading weight is 136kg

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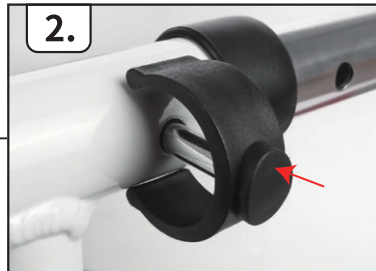
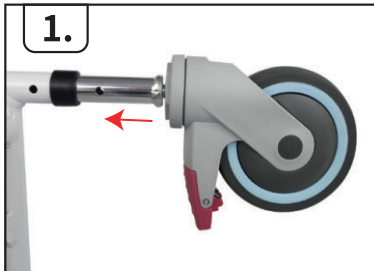
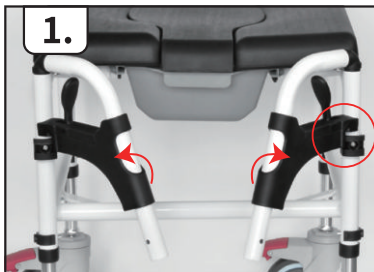
**EN**\_Insert pedal C into mainframe A, then install with pedal screws H

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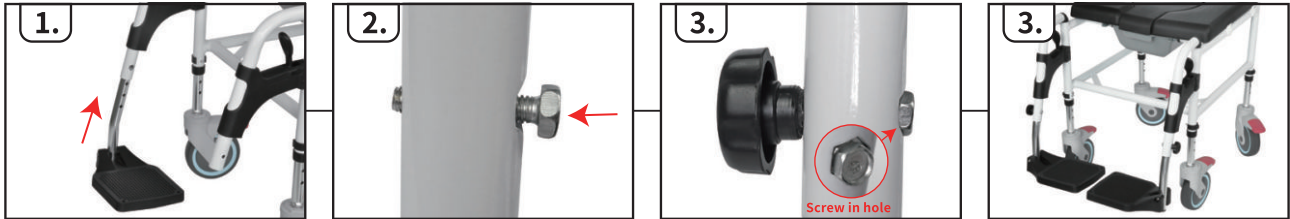
**EN**\_Insert backrest B into mainframe A and fasten with backrest screws J

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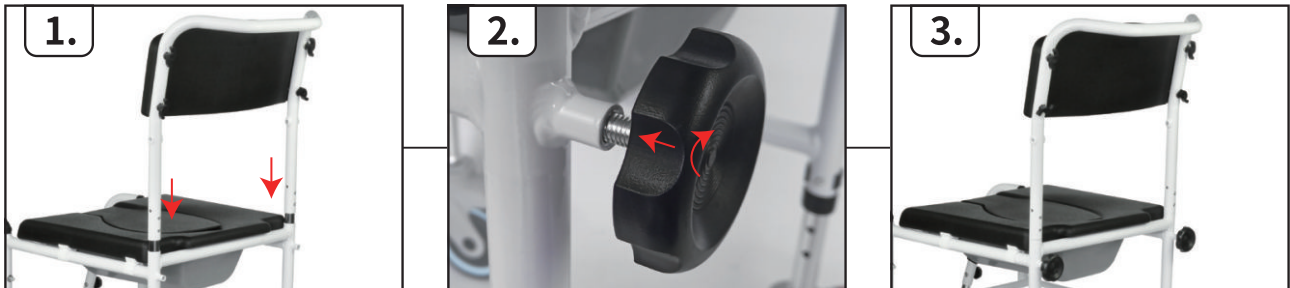
**EN\_1.** Install armrest E on backrest B with armrest screws K and tighten with the installation tool M

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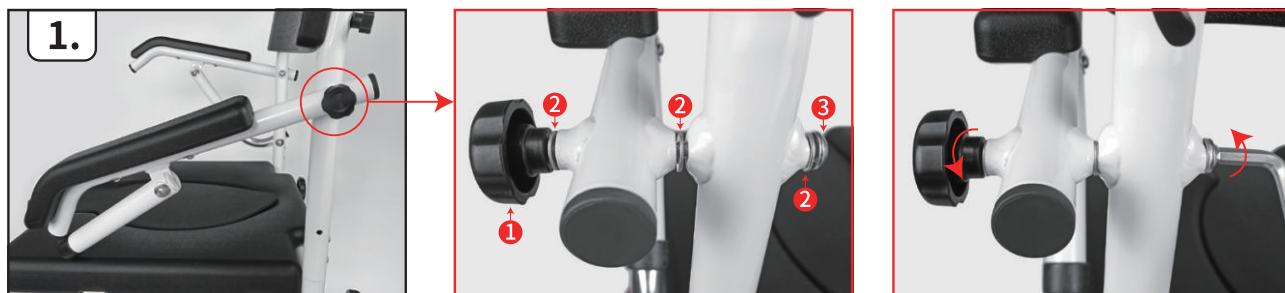
**EN\_2.** Insert armrest tube F into armrest E, Install armrest tube F on backrest B with armrest tube screws L and tighten with the installation tool M

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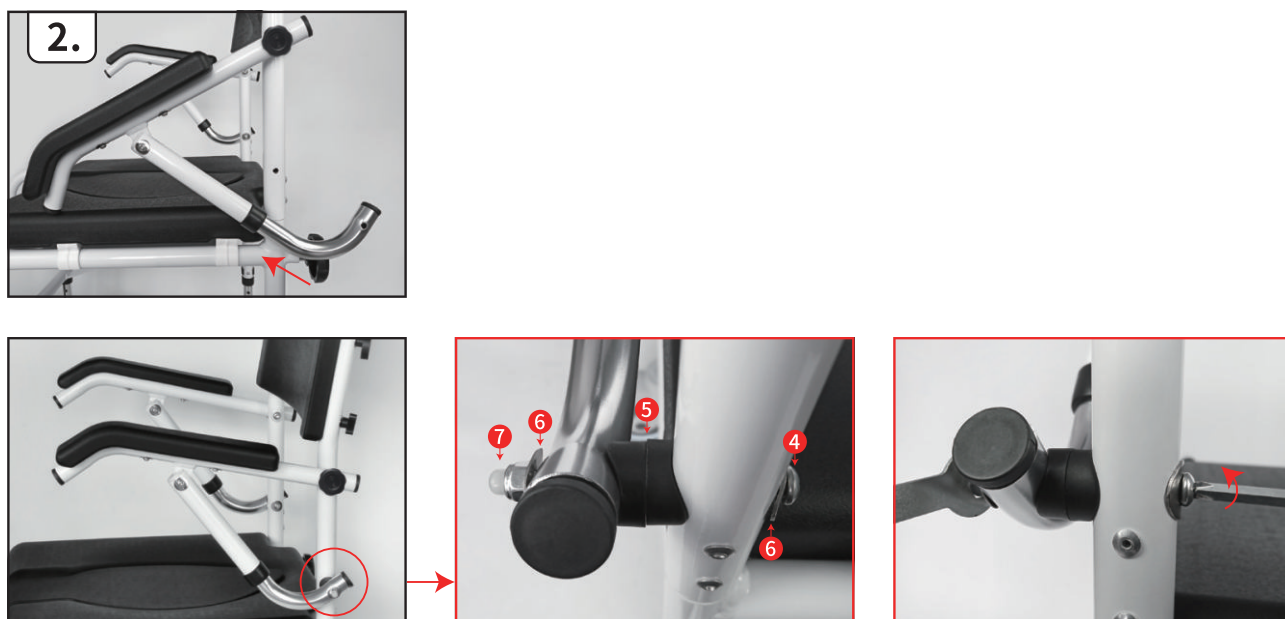
**US**

Imported by Aosom LLC  
27150 SW Kinsman Rd Wilsonville, OR 97070 USA  
MADE IN CHINA

**CA**

Imported by Aosom Canada Inc.  
7270 Woodbine Avenue, Unit 307, Markham, Ontario Canada  
L3R 4B9  
MADE IN CHINA

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**UK**

IMPORTER ADDRESS:  
MH STAR UK LTD  
Unit 27, Perivale Park,  
Horsenden lane South  
Perivale, UB6 7RH  
MADE IN CHINA

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MH Handel GmbH