EN_IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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
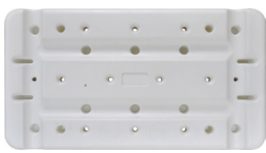
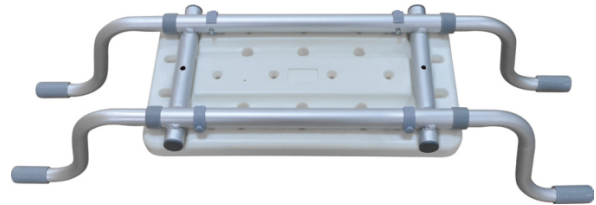
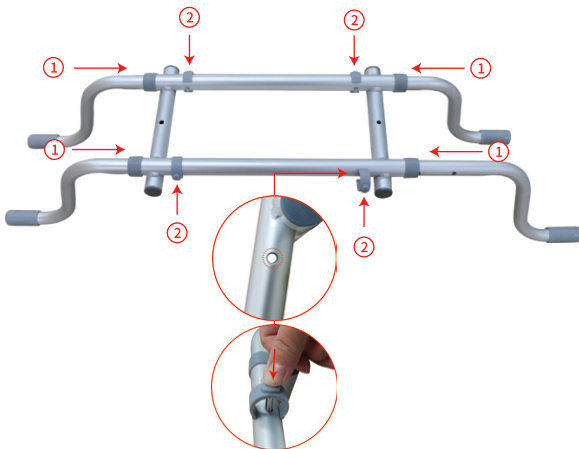
x 2

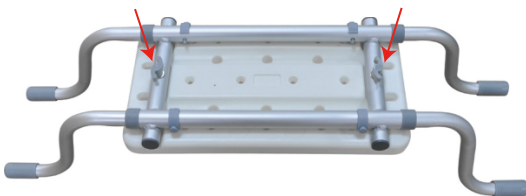x 2

Carefully read the following instructions for use

Only use this item in the manner described and for the purpose for which it is intended. Store these instructions for use carefully.

Maximum load: 150kg

Dimension: 22 x 18H cm

Adjusting the mounting width: 73.5-82.5cm (+/- 1-2cm)

Seat length: 40.5 cm

Seat width: 22 cm

Purpose

The bathtub seat board is designed to provide a stable seat when bathing in a standard bathtub. It is designed for people who have problems to sit down and / or rise from the bottom of the bathtub.

WARNING

1. The seat board cannot be mounted on the bathtub when the rubber pads, located on the side of the brackets, do not have full contact with the vertical (or almost vertical) wall of the bathtub, as shown in Fig.2.
2. Always make sure that the seat board is attached properly before using it.
3. Please check the condition of anti-slip tube cap before each using.
4. This product is only for indoor use.
5. This product is not suitable for saunas or heat cabins.
6. This product is not suitable for rehabilitation or medical treatment of persons.

Safety advice

• “Smooth or damp surfaces may impair anti-slip properties”

Do not step on seat.

• Do not lift together with person.

• Note: not for use in the rehabilitation or medical treatment of persons.

• Service life for supporting components made of plastic material is 2 years.

Assembly

1. Slide brackets and align the holes for bolt. There are two holes on the rectangle fitting for length adjust, please ensure the two hands can be fixed well on the edge of tub. **WARNING!** Please ensure the bolt is completely inserted and lock the hands.

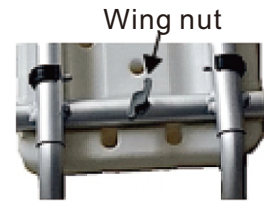
Fig.1

2. Place the rectangle fitting on the backside of seat board, align the holes for screw.

3. Screw down the screws with gasket tightly and ensure no sway between them. **WARNING!** If, on the one hand, the seat board protrudes significantly beyond the edge of the bath tub, it can be dangerous to the user and threaten to fall.

4. Check that the seat board is securely locked before use. The seat board cannot be placed asymmetrically relative to the bathtub edges, and the protections must not adhere or only partially adhere to the bathtub (Fig. 2).

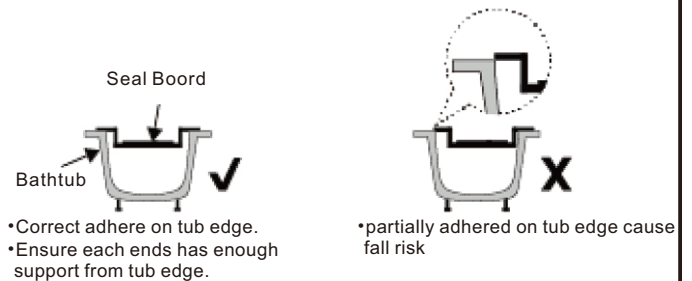
Fig.2

5. The above procedure should be used each time when mounting the seat board on the bathtub.

WARNING! Always before using the seat board make sure that it is correctly and stably mounted to the bathtub.

Cleaning and maintenance

1. It is recommended to regularly check the seat board for any damage or visible signs of wear.
2. Regular cleaning is recommended. The seat board should be cleaned with a soft cloth and soapy water after each use it off. **WARNING!** Do not use solvents, bleaches or abrasives to clean the handle.

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If you have any questions, please contact our customer care center, Our contact details are below:

US

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC
27150 SW Kinsman Rd Wilsonville, OR 97070 USA
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

UK

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:
MH STAR UK LTD
Unit 27, Perivale Park,
Horsenden lane South
Perivale, UB6 7RH
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

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 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.
7270 Woodbine Avenue, Unit 307, Markham,
Ontario Canada L3R 4B9
MADE IN CHINA

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