DE: MONTAGEANLEITUNG

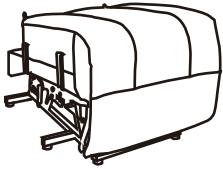
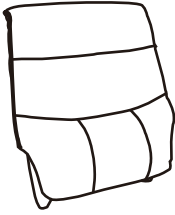
EN: ASSEMBLY INSTRUCTION

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IN CHINA HERGESTELLT

We're excited for you to regain your independence with Homcom! To ensure the best experience, please read this guide carefully before using your new lift chair.

Be sure to keep this manual close by, preferably in your chair pocket so that you can always refer to it as needed.

Safety Rules	4
Lift Chair Parts	5
Assembly Instructions	6
Lift and Recliner Function.....	8
Functional Description Of Massager	9
Troubleshooting	12

SAFETY RULES

- Never set the lift chair in wet/damp areas or where dust can accumulate as these could cause an electrical fault or a mechanical dysfunction.
- Never use the lift chair inside your home if the room temperature is over 40°C.
- Never place the lift chair near a heat source (oven, chimney, furnace etc...) or in direct sunlight.

Do Not use the lift chair if:

- The lift chair is wet.
- Your hands are wet.
- Only clean the chair with a dry cloth.

Situations where you should not use the lift chair:

Switch the lift chair's system off if it gets wet or seems not to be working properly, if any part of the lift chair is damaged or malfunctioning, if any part of the lift chair appears to be damaged or wiring is exposed.

Usage:

- Indoors, avoid high temperatures and wet or damp areas.

Precaution:

- Never sit on the remote or place heavy objects on top of it.
- Do not drop the remote on the floor.
- Do not tug on the electric cable.
- Do not use the remote with wet fingers.
- Do not press or touch the buttons with sharp objects.
- Never unplug the remote without first turning the power off.
- The lift chair should rest for 30 minutes after it has been used for 30 minutes.

When the system of the lift chair is not in use:

- Make sure it is switched off.
- Unplug the lift chair's system from the wall outlet when you are not going to use it for long periods of time.
- Keep the lift chair in a clean and dry place.

Disassembling the massage armchair:

- Do not remove the back of the armchair .
- Do not try to disassemble or repair the armchair yourself.

Correct way to handle the mains plug:

- Check that you have the right voltage.
- Never use force on the cable or plug.
- Never pull the cable with wet hands.
- When removing the plug, hold the plug itself, do not pull on the cable.
- Do not use the armchair massage system if either the cable or the plug are damaged.

The Parts forming the massage armchair:

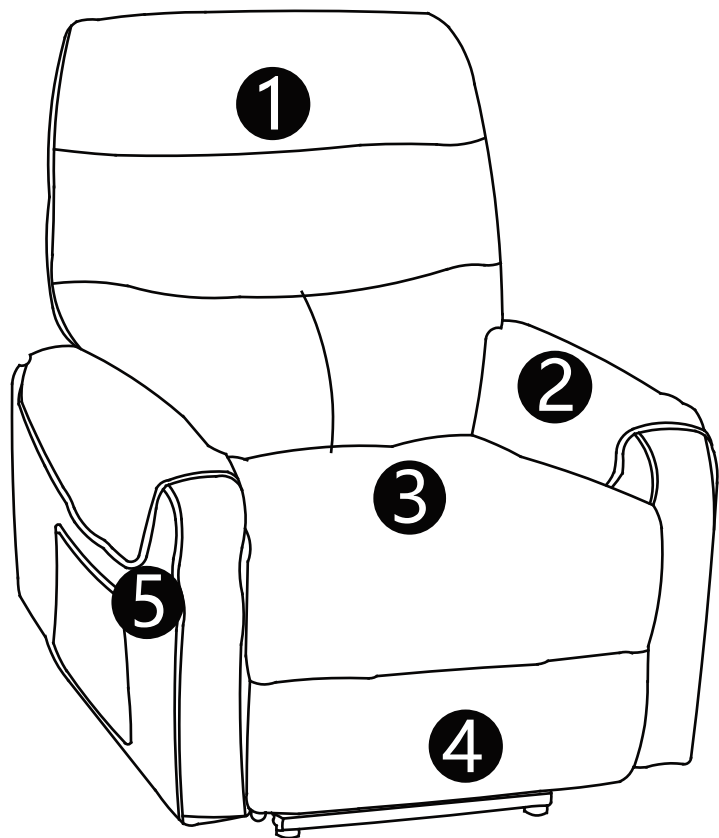
1)Back

2)Armrests


3)Seat

4)Footrest


5)Control unit


PARTS




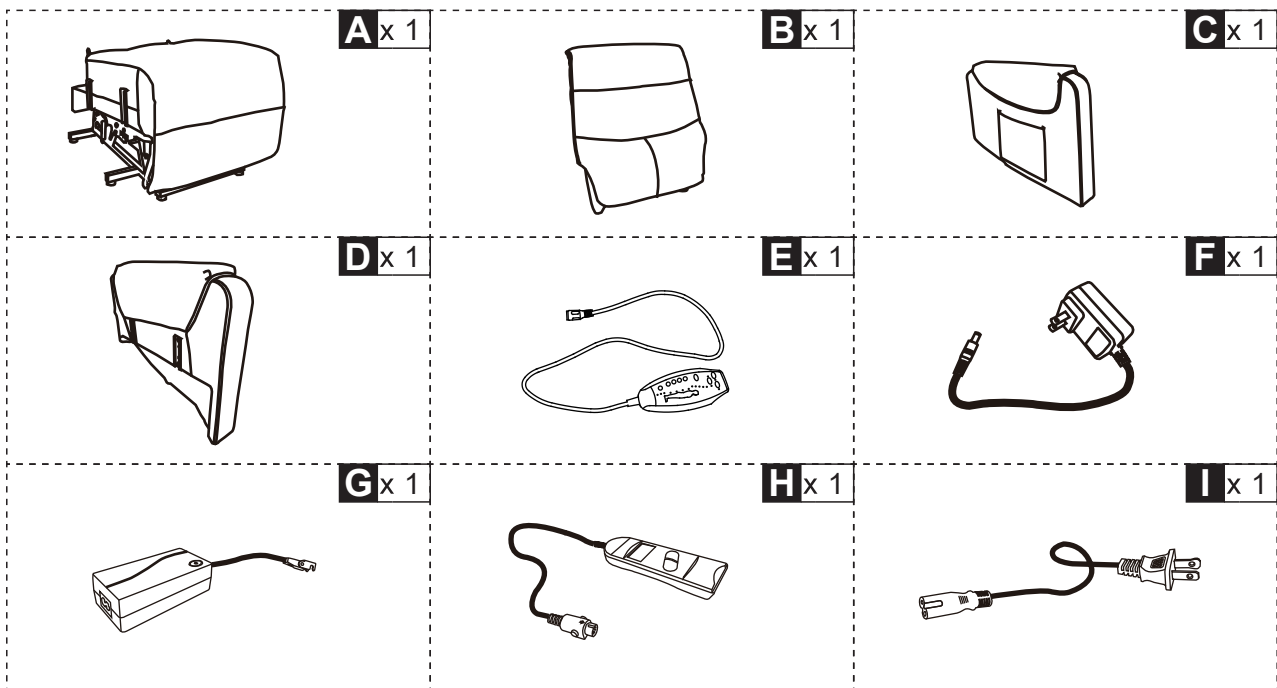
HARDWARE

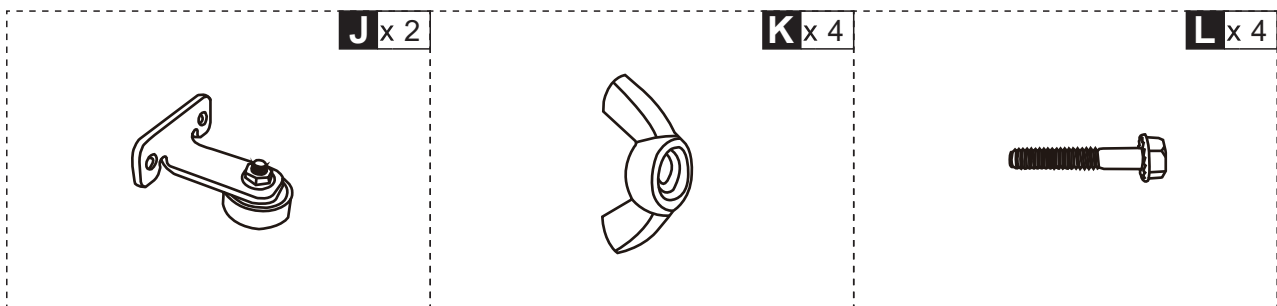01

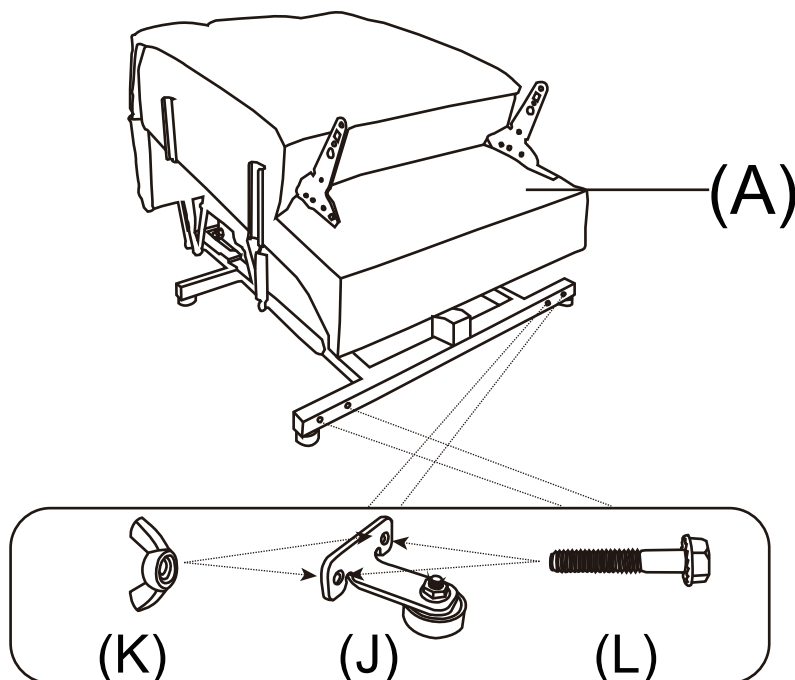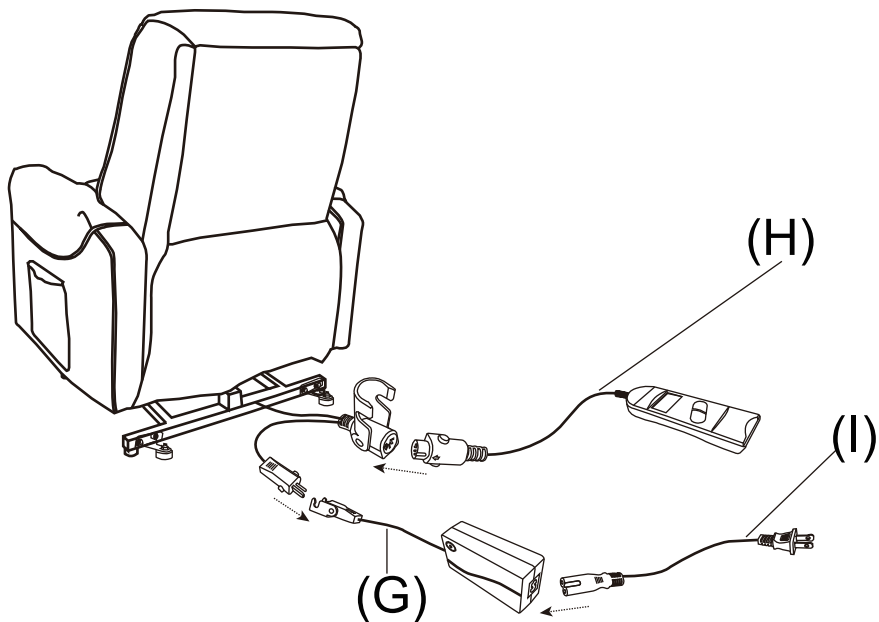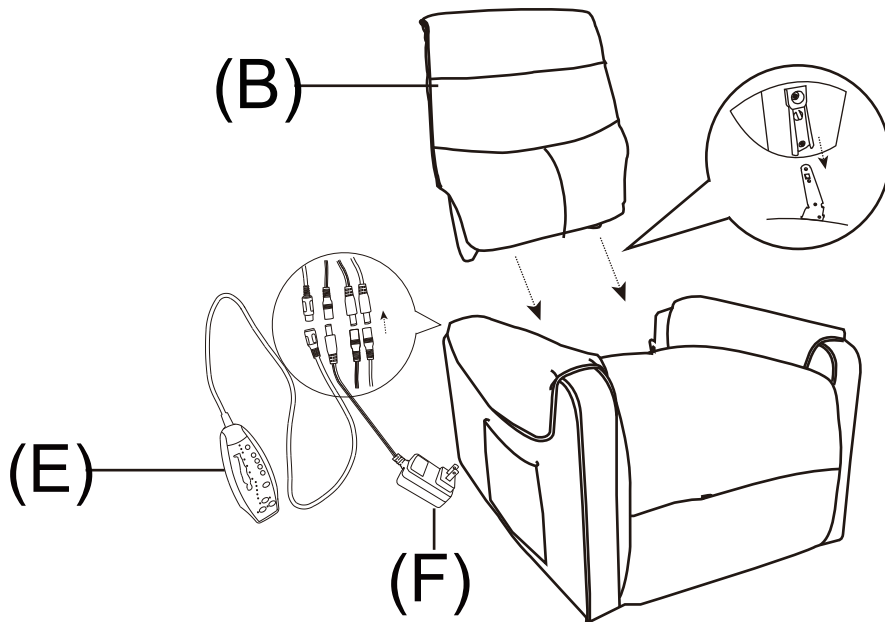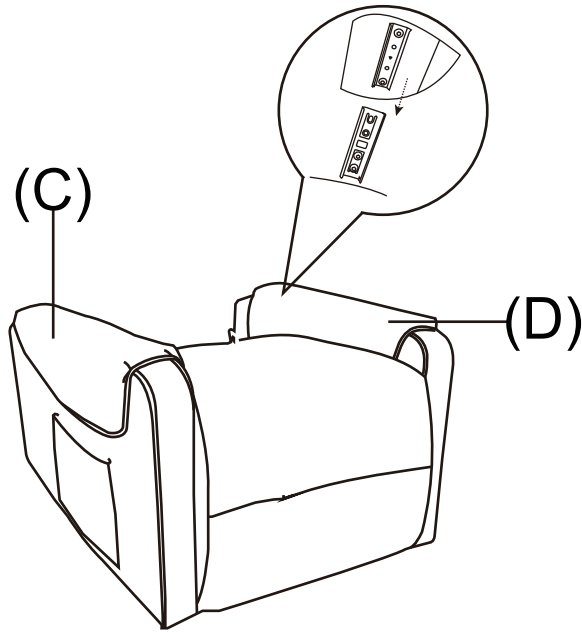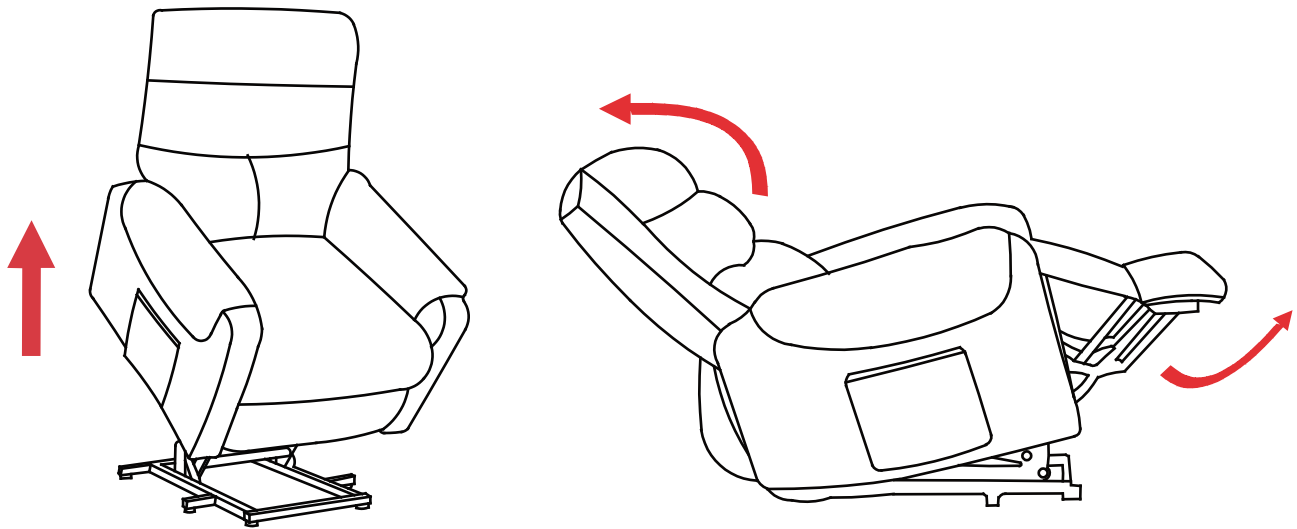Lift And Recliner Function

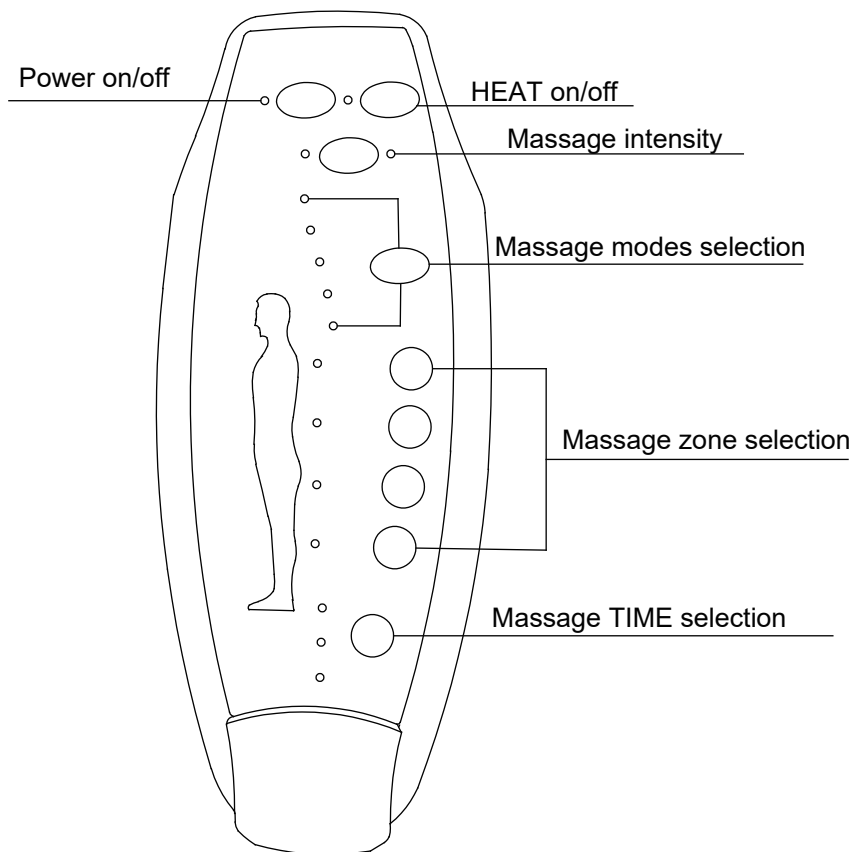
- Pressing the right button, the back will move down and the footrest will rise up. And if you press the left button the Lift chair will return to normal position, after that continue to press the left button Lift chair will rise up smoothly.
- To stop the chair at any time simply release the button that is being pressed.

Warning

Never press both buttons together as this will damage your chair.

Never stand up while the chair is still working in order to avoid an accident.

FUNCTIONAL DESCRIPTION OF MASSAGER


PRODUCT FEATURE

- 8 Vibration Motors
- Eight Massaging modes
- Five Adjustable intensity
- Four massage zones, each zone can be operated individually
- The default timing is 15-30 min

HOW TO USE

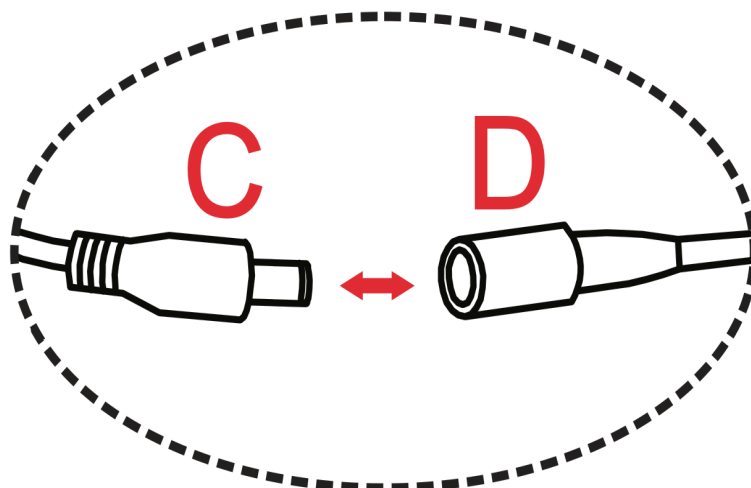1. Insert adapter jack into the socket found on back of the massage chair, then plug the adapter into a wall outlet.

2. Turn the unit ON by pressing the ON/OFF button once, The LED will illuminate when power is on.
3. Turn on the massage in each zone by pressing the Massage zone. LEDs will illuminate when each zone is activated. To turn off a zone, press the button again.
4. To choose a desired massage mode, press the MODE button. To change modes, press MODE button until LED for desired mode illuminates
5. To change the massage intensity press the +/- button.
6. NOTE: Discontinue use immediately if you feel any discomfort or pain.
7. NOTE: for your benefit, the massage chair is equipped with an auto shut-off feature that will turn the chair unit off after 15 /30minutes of continuous use.
8. DO NOT use for more than 30minutes of continuous use.

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following:

1. This product is for use on a 12V circuit Make sure that the product
2. Only use this product with original power supply type nr : IVP1200-0500
3. Always unplug the massager from the power source after use, and before cleaning.
4. Do not reach for massager if it has fallen into water. Unplug immediately.
5. An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, before putting on or taking off parts or attachments.
6. Do not use outdoors.
7. Do not carry this appliance by supplied cord as a handle.
8. Keep the cord away from heated surfaces.
9. Close supervision is necessary when this appliance is used by on or near children, invalids or disabled persons.
10. This unit is not a toy. Children should not use it or play with it.
11. Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
12. Do not use near eyes or other highly sensitive areas.
13. To disconnect, push "POWER" key to "OFF" place, make sure all functions stopped, then remove plug from outlet.
14. Keep cord away from heated surfaces.
15. Use this appliance only for its intended use as described in this manual. Do not use attachments not provided with the unit.

HEALTH CAUTION FOR MASSAGE CHAIRS

- If you have any concerns regarding your health, consult your doctor before using this product.
- Individuals with pacemakers should not use this product.
- If you experience pain in a muscle or joint for a prolonged period of time. Discontinue use and consult your doctor. Persistent pain could be a symptom of a more serious condition.

MAINTENANCE INSTRUCTIONS

TO STORE:

Place unit in its box or safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric. Do not wrap power cord around the unit. Do not hang the unit by power cord.

TO CLEAN:

- Insure the unit is unplugged from the power source, and allow to cool before cleaning.
- Do not place in any liquid to clean. Clean fabric cover by wiping with a damp sponge, using a mild detergent or suds. Pat dry with clean, dry cloth.
- Do not use abrasive cleaners, brushes, gasoline, kerosene, glass /furniture polish, paint thinner or water hotter than 120*to clean the unit.
- To clean the unit, make sure the massager is unplugged. Allow the unit to cool and the wipe down with a damp clean cloth. Do not use any strong chemical cleaners.

TROUBLESHOOTING

If the lift chair is not working, check the following chart. Make sure you turn the power off and unplug from the wall outlet.

If you have any questions, please contact us. Do not disassemble or repair any part of the lift chair yourself.

PROBLEM

POSSIBLE CAUSE

The lift chair does not function when the button on the control unit is activated.

Check that the plug is pushed into the wall outlet properly. Check whether the power supply button is on.

	115V	230V	
Input voltage	100-240	100-240	V
Input AC frequency	50-60	50-60	Hz
Output voltage	29	29	V
Output current	1.8	1.8	A
Output power	52.2	52.2	W
Average active efficiency	88.21	88.43	%
Efficiency at low load (10 %)	88.21	88.43	%
No-load power consumption	0.041	0.112	W