EN_ IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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EN

Dear customer,

Thank you for purchasing this product. So that your appliance serves you well, please read all the instructions in this user's manual. If you have any questions, please contact our customer care center, Our contact details are below:

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EN

Maintenance

1. The exterior of this rollator walker should be cleaned periodically with a soft damp cloth. Do not use abrasive cleaners or caustic chemicals.
2. Check the parts for any looseness, wear, or damage periodically. If any problems are found, correct them before using them.
3. If haven't used it for a long time, please store it in a cool, dry and safe place.

Warning

1. Before using, make sure the rollator is fully opened.
2. Checking the brakes and seat are locked, seat part especially.
3. The brakes must be locked before using the seat.
4. Max loading weight is 330LBS

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A x 2

B x 2

EN_1. Insert two rear wheels into the mainframe①
2. Make sure the snap button is securely locked

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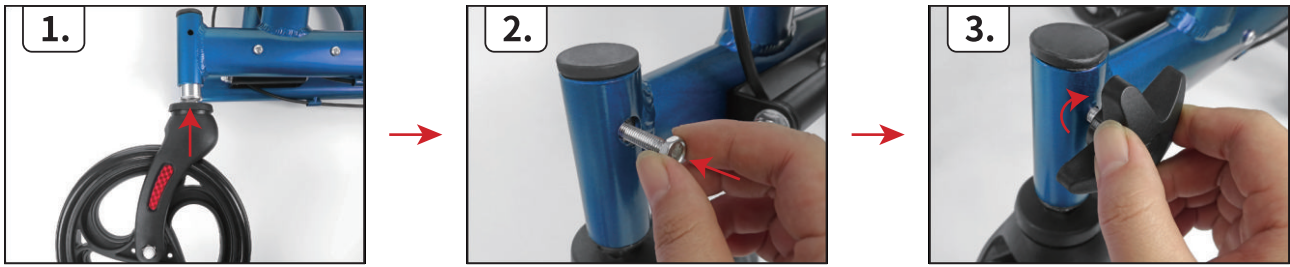
- EN_1.** Insert two front wheels② into the mainframe①
 2. Insert the screws into the holes, after making sure the holes are aligned
 3. Put the "star bolt" in the screw and tighten it

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EN_1. Hold both sides of the seat and push down it until the seat is completely flat
 2. Press the cushion down until the hook hooks in (the hook is under the seat)

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EN_1. Press the lock, insert two armrests into mainframe
 2. Make sure that lock

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MH Handel GmbH