EN_ IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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EN**Maintenance**

1. The exterior of this rollator walker should be cleaned periodically with a soft damp cloth. Do not use abrasive cleaners or caustic chemicals.
2. Check the parts for any looseness, wear, or damage periodically. If any problems are found, correct them before using them.
3. If haven't used it for a long time, please store it in a cool, dry and safe place.

Warning

1. Before using, make sure the rollator is fully opened.
2. Checking the brakes and seat are locked, seat part especially.
3. The brakes must be locked before using the seat.
4. Max loading weight is 150kg

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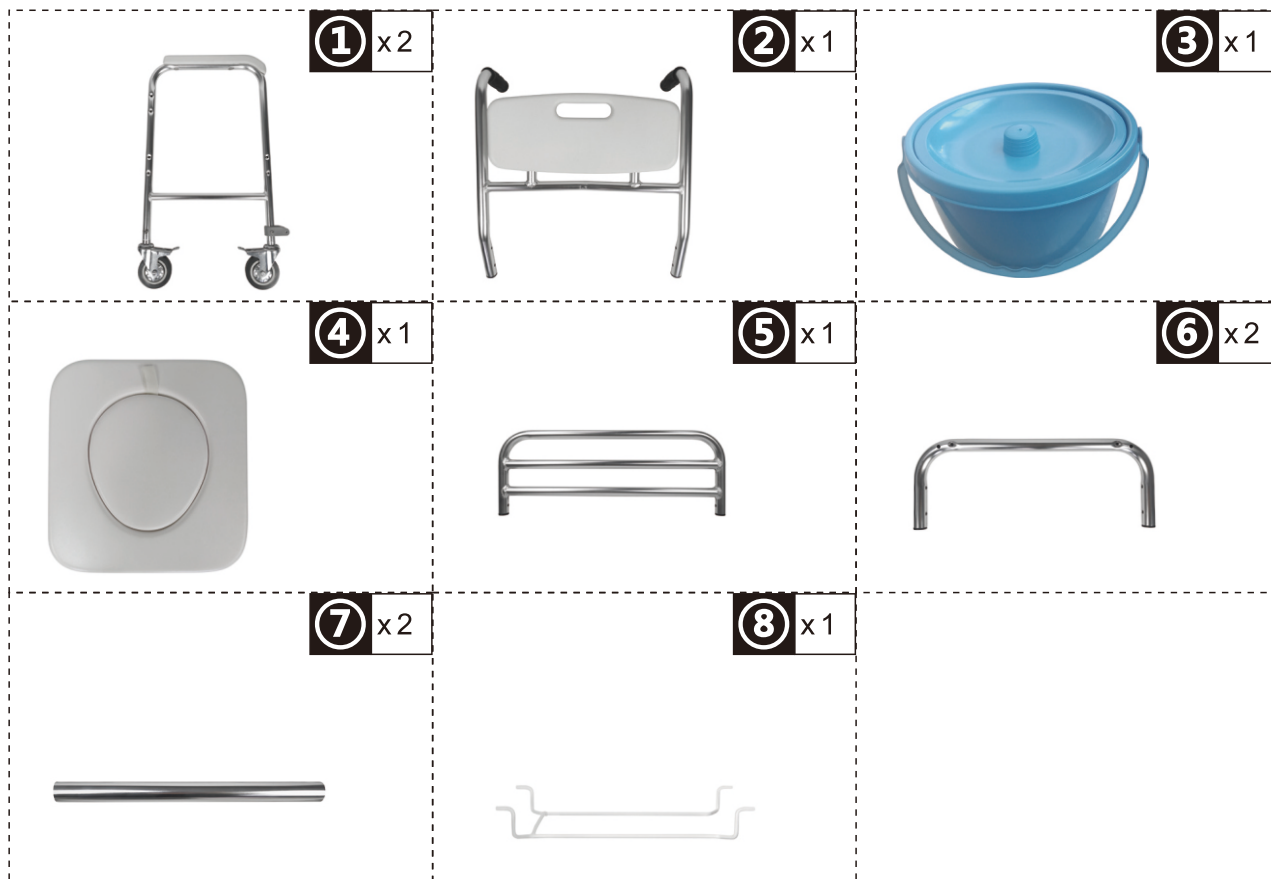
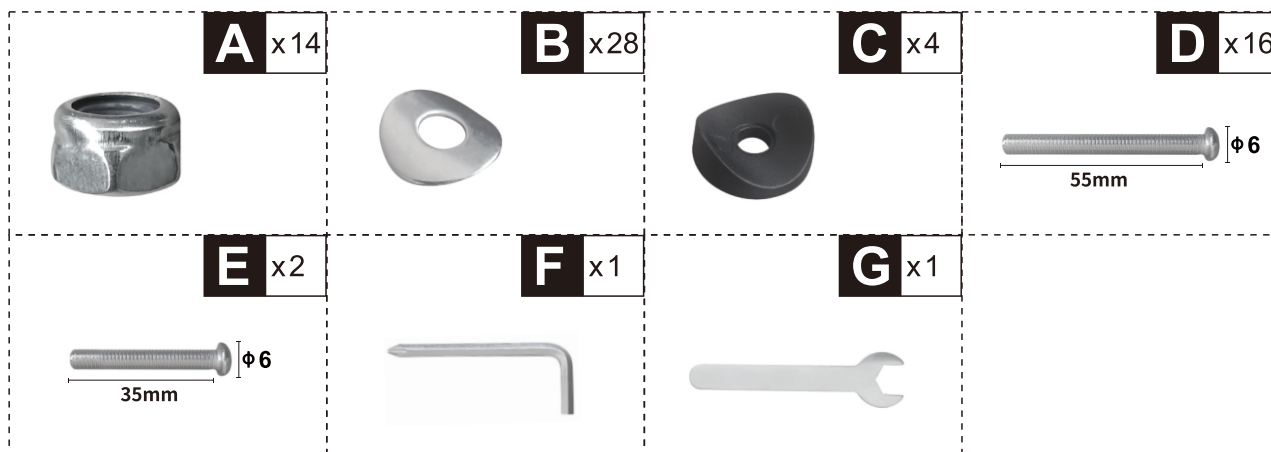
HARDWARE

NOTE

EN_Do not confuse similar screws.Do not tighten bolts fully until all parts are connected!

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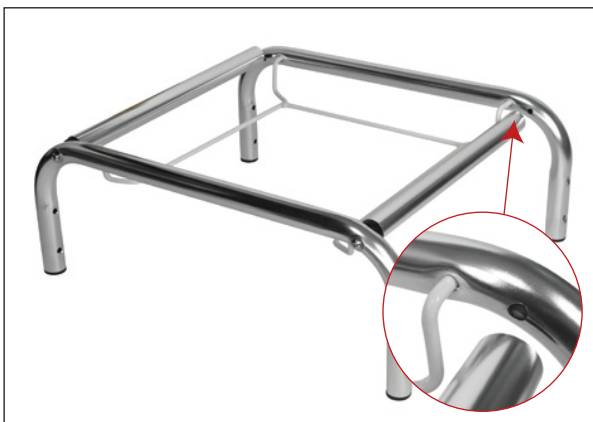
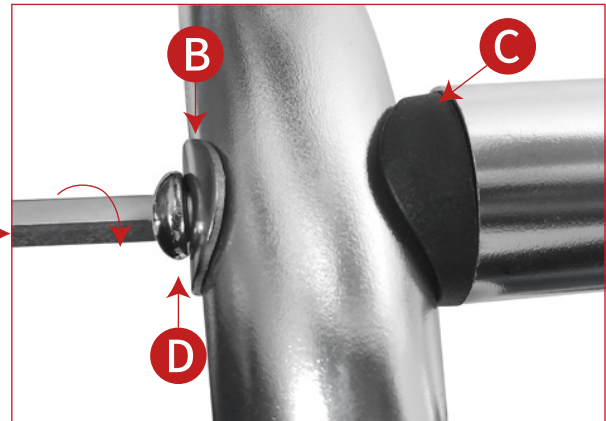
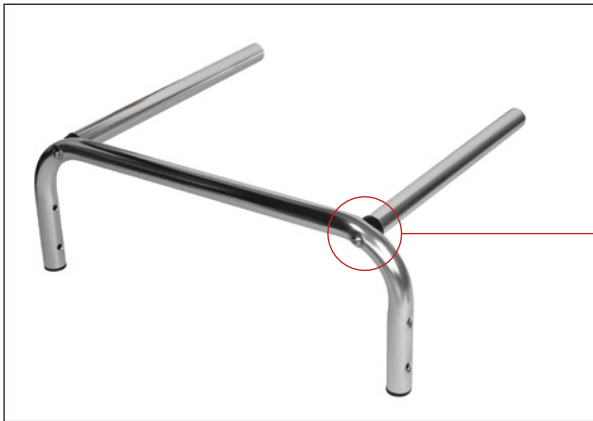
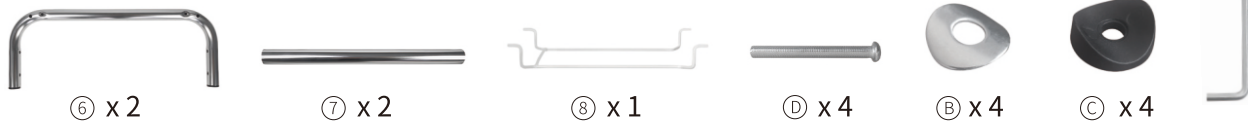
EN Joint ALUMINUM FRAME (7) to ALUMINUM FRAME (6), with 1PC SCREW (D) 1PC PLASTIC GASKETS (C) and 1PC METAL GASKETS (B) in each side, insert Bucket Hanging Bracket (8) into ALUMINUM FRAME (6), following Previous step again.

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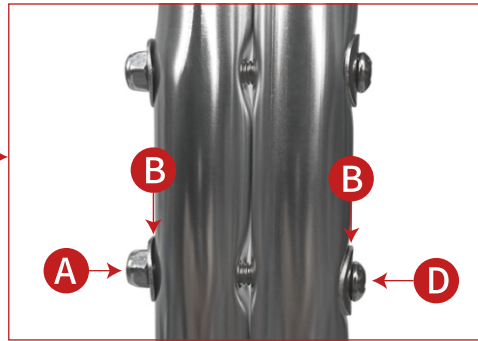
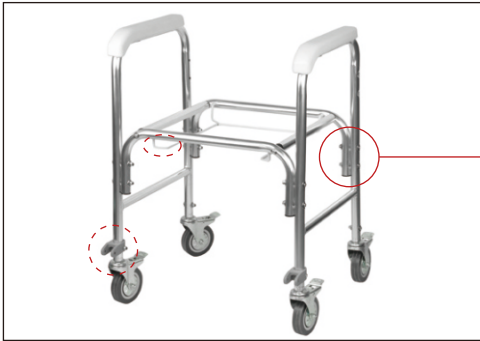
② x 16

③ x 8

EN Joint two ARMRESTS (1) to ALUMINUM FRAME (6) , with 1PC SCREW (D) , 2PCS METAL GASKETS (B) and 1PC SCREW NUT (A) in one hole of each side (each side has two holes).

Notes: Please make sure the semi-closed bracket face front to avoid bucket falling down.

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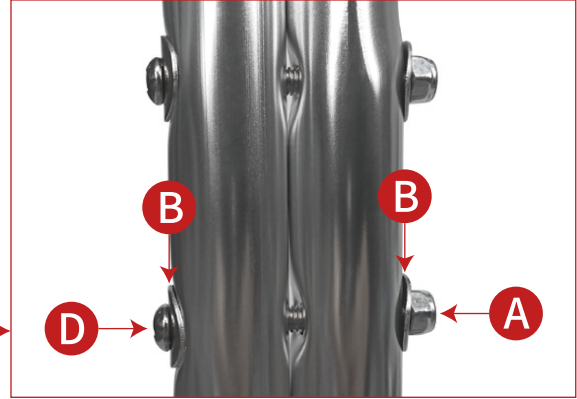
EN_Joint two ARMRESTS (1) or BACKREST WITH HANDLES (2), with 1PC SCREW (D), 2PCS METAL GASKETS (B) and 1PC SCREW NUT (A) in one hole of each side (each side has two holes).

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③ x 1

EN Put BUCKET (3) on the BUCKET HANGING BRACKET (8).

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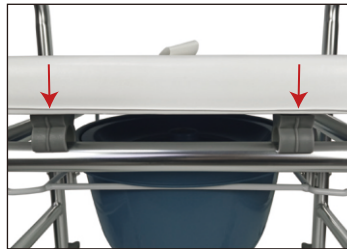
④ x 1

EN Put PADDED SEAT (4), jointing ALUMINUM FRAME (6) to two buckles.

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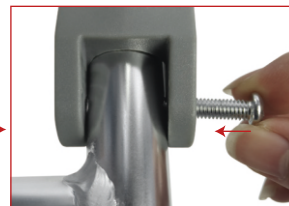
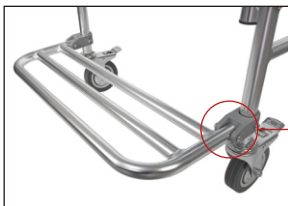
⑥ x 2

④ x 2

EN Put PEDAL (5) on the anchor of ARMRESTS (1), align with the hole, insert the screw and install the nut on the other side

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If you have any questions, please contact our customer care center, Our contact details are below:

US

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC
27150 SW Kinsman Rd Wilsonville, OR 97070 USA
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

CA

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.
7270 Woodbine Avenue, Unit 307, Markham,
Ontario Canada L3R 4B9
MADE IN CHINA

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If you have any questions, please contact our customer care center, Our contact details are below:

UK

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:
MH STAR UK LTD
Unit 27, Perivale Park,
Horsenden lane South
Perivale, UB6 7RH
MADE IN CHINA

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