IN240500143V01\_GL

713-169V00\_713-169V01\_713-169V02

**EN\_IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.**

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## Aluminum Telescoping Wheelchair Track Ramps

### General Guidelines

- It is the user's responsibility to read and follow all instructions.
- Keep these instructions with the product at all times and review before each use.
- It is the responsibility of this product's owner to furnish the instructions to any person that borrows or purchases the product.
- Inspect the product before use to ensure it is assembled properly and all parts are in safe working order and free of defects.
- Never modify this product in any way.
- All circumstances cannot be addressed in these instructions. Please use common sense and practice general safety measures when using this product.
- Parts and/or instructions are subject to change without notice.

### Safety

#### MAXIMUM \*OCCUPIED / \*\*UNOCCUPIED HEIGHT LIMIT TO GROUND LEVEL

- 2' ramp: Occupied loading: 4" maximum, unoccupied loading height: 6"
- 3' ramp: Occupied loading: 6" maximum, unoccupied loading height: 9"
- 4' ramp: Occupied loading: 8" maximum, unoccupied loading height: 12"
- 5' ramp: Occupied loading: 10" maximum, unoccupied loading height: 15"
- 6' ramp: Occupied loading: 12" maximum, unoccupied loading height: 18"
- 7' ramp: Occupied loading: 14" maximum, unoccupied loading height: 21"
- 8' ramp: Occupied loading: 16" maximum, unoccupied loading height: 24"
- 9' ramp: Occupied loading: 18" maximum, unoccupied loading height: 27"
- 10' ramp: Occupied loading: 20" maximum, unoccupied loading height: 30"

\*Occupied is a person sitting in the wheelchair/scooter while loading (2:12 slope or 9.6 degree angle)

\*\*Unoccupied is loading with nobody sitting in the wheelchair/scooter (3:12 slope or 14.5 degree angle)

MAXIMUM WEIGHT CAPACITY IS 600 lbs. (Weight rating is based on evenly distributed load on 2 axles.)

### Warning

- NEVER exceed the rated capacity of your ramp.
- NEVER exceed the maximum height limit listed above.
- Use caution at all times when loading and unloading.
- ONLY load and unload onto a stable, level surface.
- NEVER stand in the path of equipment or vehicle. Keep bystanders and children away.
- DO NOT leave ramps unattended.
- Beware of pinch points. Keep all body parts clear of pinch points when extending or shortening the ramps.
- Ramp surfaces will have less traction when wet. Use caution.
- Always shorten and secure ramps with the latches before carrying.
- The ramp is not a toy. DO NOT use ramp as a jump, scaffold, bridge, ladder, or anything other than its designed purpose.
- ALWAYS make sure the top of the ramp attaching lip is properly secured onto the step, landing, or vehicle before using.
- ALWAYS follow manufacturer recommendations for chair or scooter.

### Operation Guidance

#### Loading wheelchair:

- Position the wheelchair in the direction of the ramp with front wheel forward. Roll or ride the wheel-chair slowly and steadily up the ramp, ensuring that the wheels follow the center of the ramp.
- Take extra care in wet conditions to ensure the wheels do not slide on the ramp.

**Unloading wheelchair:**

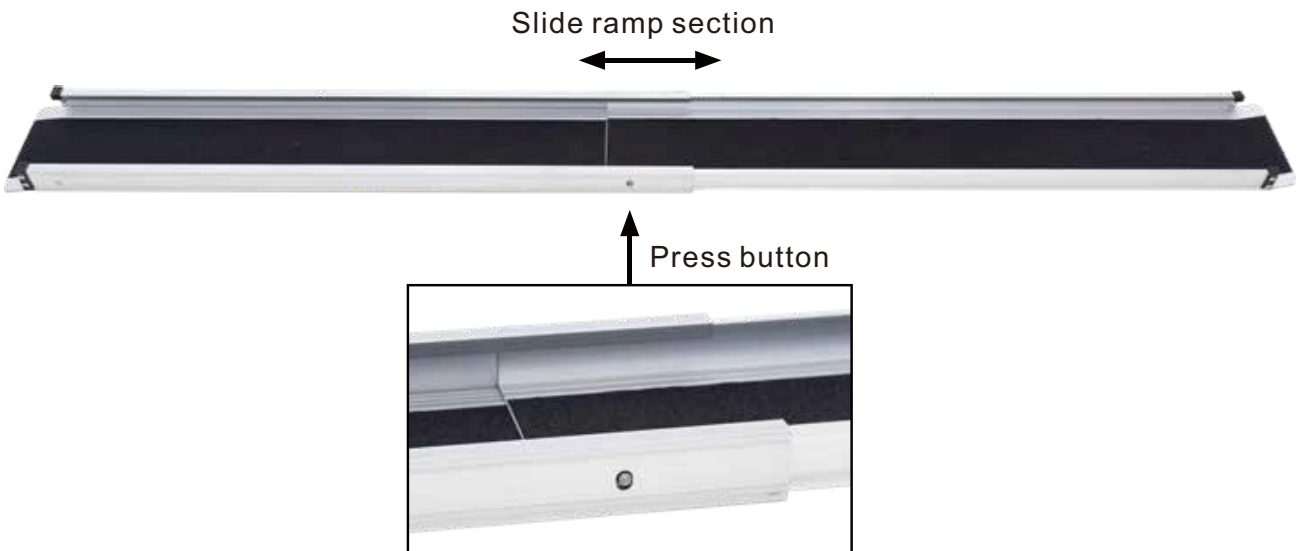
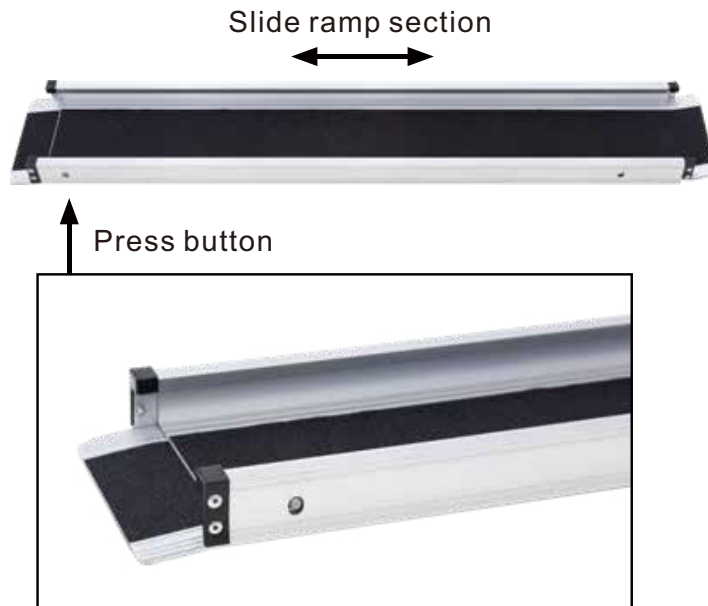
- Position the ramp as detailed above directly behind the rear wheel of the wheelchair. Allow the wheelchair to roll backwards down the ramp, ensuring the wheels follow the center of the ramp.
- Always have a qualified assistant present when using the ramp.

**Maintenance**

- It is the user's responsibility to check the loading ramp and components before each and every use. If the ramp is damaged, it should NOT be used.
- Maintain the ramp by regular cleaning.

**Extend or Shorten the Ramps**

To extend or shorten the ramps, press in the button on the side of the ramp to release it. Slide the ramp to the desired position, taking care not to pinch your fingers. The ramp will lock into the short or long position when the button fits into the hole.

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If you have any questions, please contact our customer care center, Our contact details are below:

**US**

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC  
27150 SW Kinsman Rd Wilsonville, OR 97070 USA  
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

**CA**

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.  
7270 Woodbine Avenue, Unit 307, Markham,  
Ontario Canada L3R 4B9  
MADE IN CHINA

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If you have any questions, please contact our customer care center, Our contact details are below:

**UK**

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:  
MH STAR UK LTD  
Unit 27, Perivale Park,  
Horsenden lane South  
Perivale, UB6 7RH  
MADE IN CHINA

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