INcwb027V01_US_UK_CA


712-029

IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY

ASSEMBLY & INSTRUCTION MANUAL

PUSH FORWARD TO SECURE THE BREAK

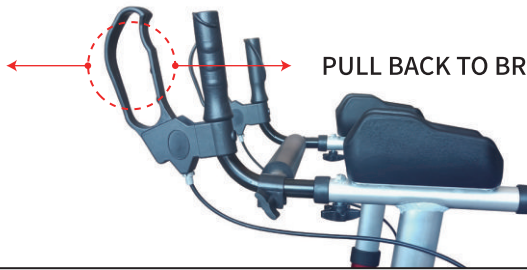PULL BACK TO BREAK

BRAKE INSTRUCTIONS

WARNING

The brakes must be in the locked position before using the seat. Do not exceed the maximum weight capacity (including bag contents). Do not attempt to push the rollator while someone is sitting on the seat. Before each use make sure the rollator frame is fully opened and in a stable position. Check the brakes for proper operation. Make sure that all parts are secure and that moving parts are in good working order.

SPECIFICATION

Weight Capacity	300lbs
Seat Pad Height	20.5"
Seat Dimensions	10" x 18"
Front/Rear Wheels	7.5"


ADJUSTABLE HEIGHT

Size	Handlebars Height
Mini	37"
Max	42"

PART LIST

Main frame

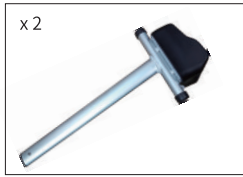Handle

Backrest

Bag

Star screw


Star screw


Screw/nut


Step 1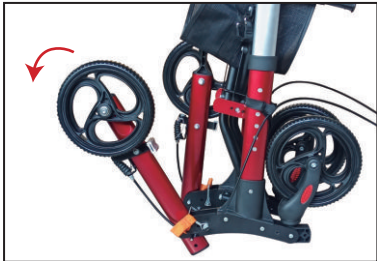

1. Unfold the rear leg.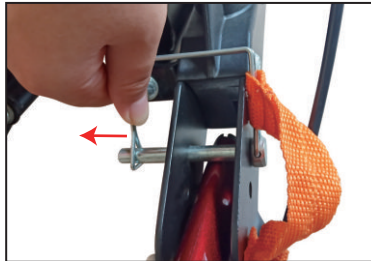


2. Take the clamps apart and take them out

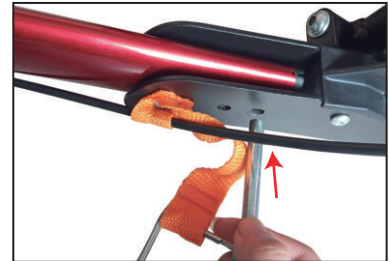
3. Insert plug into hole on outside of rollator frame.

4. Use the clip to buckle the other end of the plug.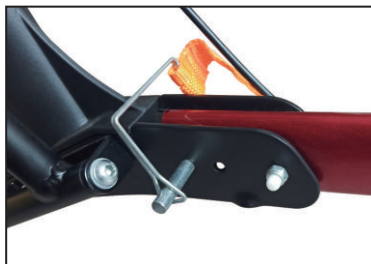

5. Installation is complete

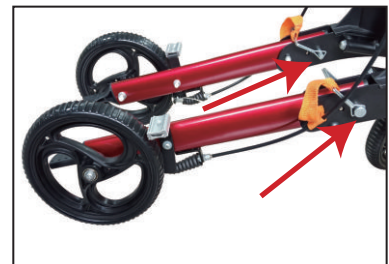
6. Congratulations on the first step of installation

Step 2

1. Unfold the front wheel.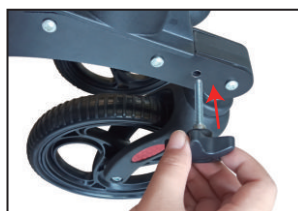

2. Insert the star screw "B" into the hole on the outside of the roller holder.


3. Tighten the star screw by hand until secure.

4. Congratulations on the second step of installation


Step 3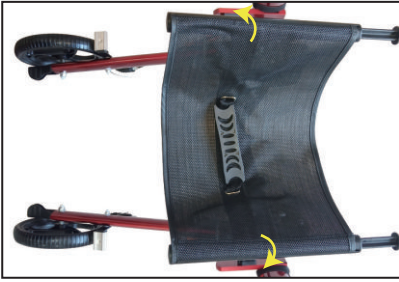

1. Hold both end of the seat and push down until the seat is completely flat.

2. Press the cushion down until the hook hooks in (under the seat)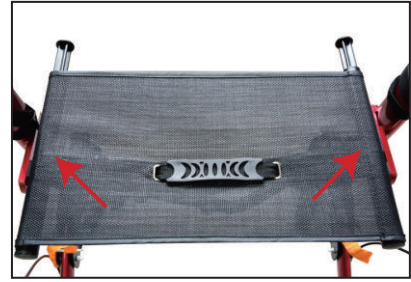

3. Congratulations on the third step of installation

Step 4


1. Insert the two height adjustable handles into the main frame (note that the brake lines correspond to the left and right)

2. Push button to adjust desired height.


Step 5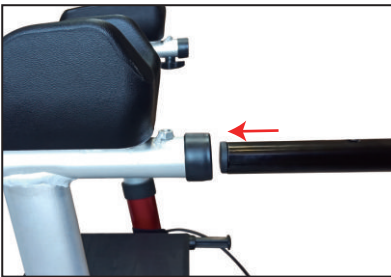


1. Insert hand grip into appropriate hole

2. Insert the star screw "A" into the hole from bottom up

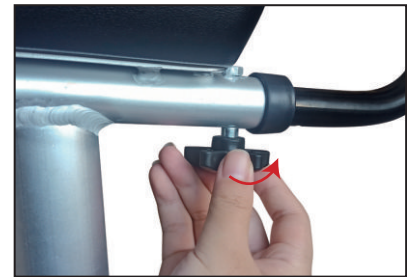
3. Tighten the star screw by hand until secure.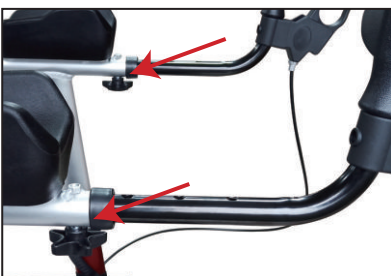


4. Congratulations on the fifth step of installation


Step 6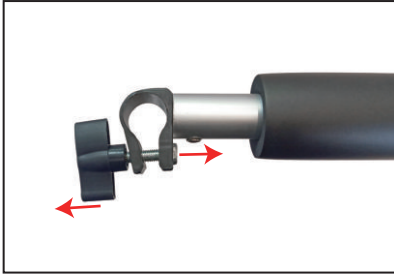


1. Remove the screw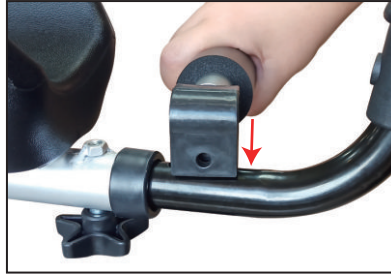


2. Insert backrest on the arm of the brakehandle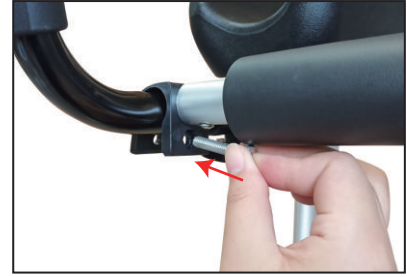

3. Insert the screws "C" from the inside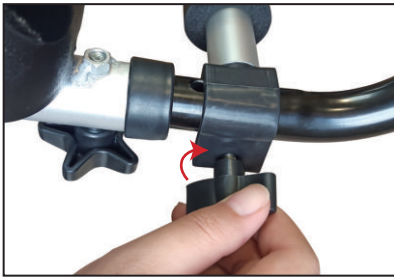

4. Tighten the nut by hand until secure.

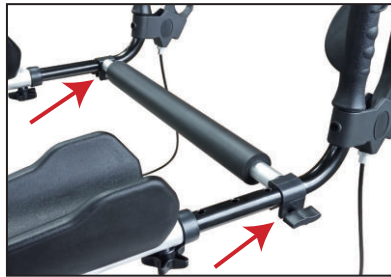5. Congratulations on the Sixth step of installation

Step 7

1. After pushing the cart seat flat, take out the bag.

2. The two sides of the belt into the chair.

PULL IT UP, AND YOU CAN FOLD IT

When the brake resistance is not enough, loosen the screw to adjust the brake line

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