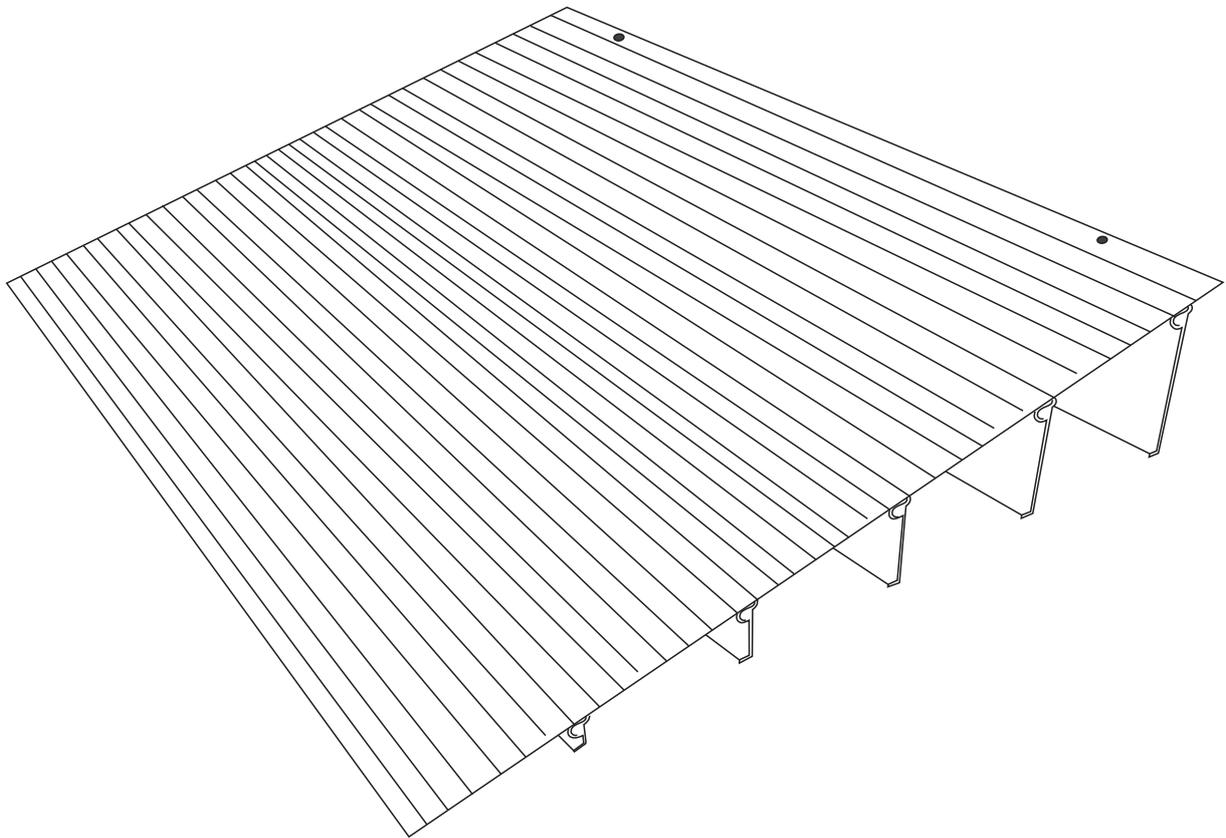
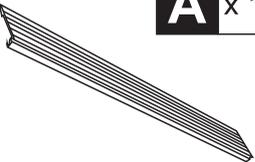
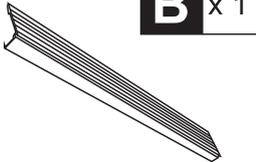
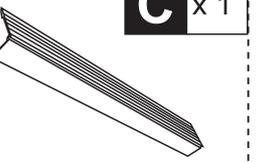
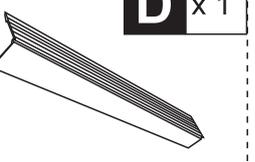
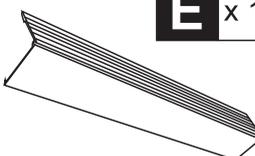
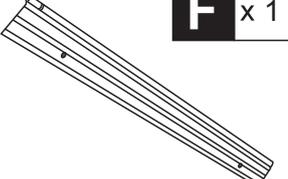
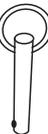
INcwc005V01_US_UK_IT

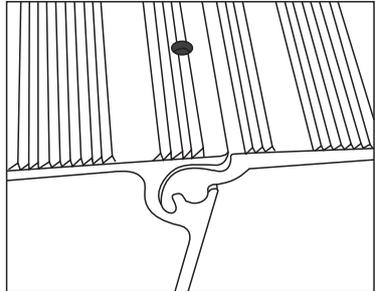
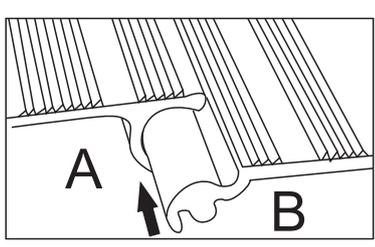
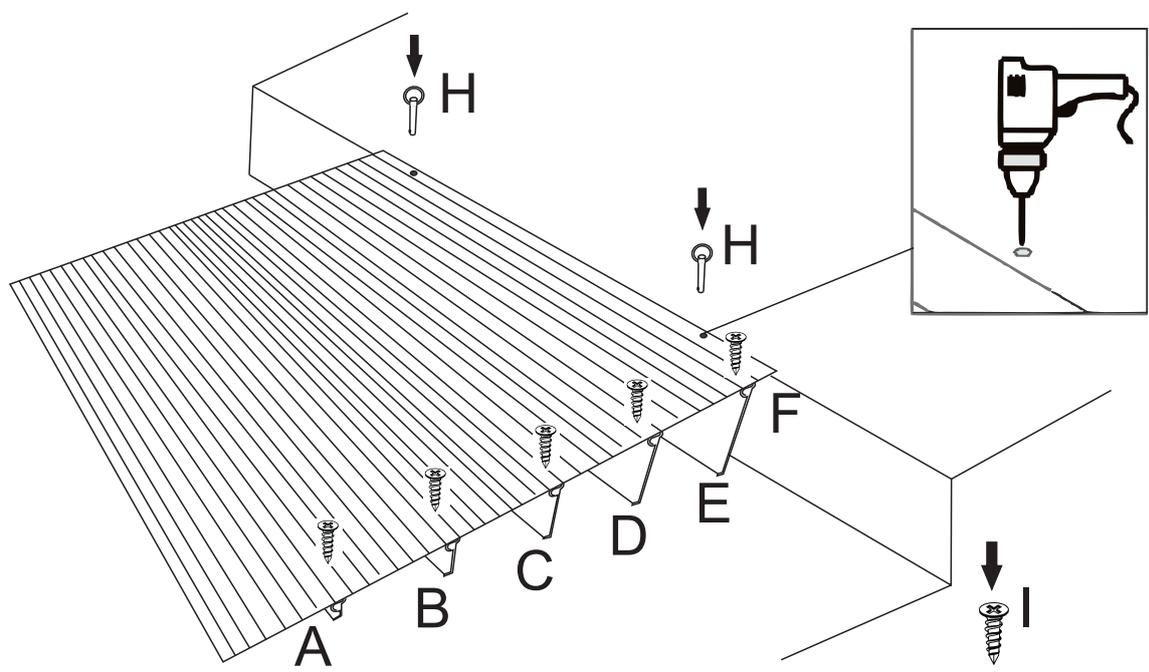
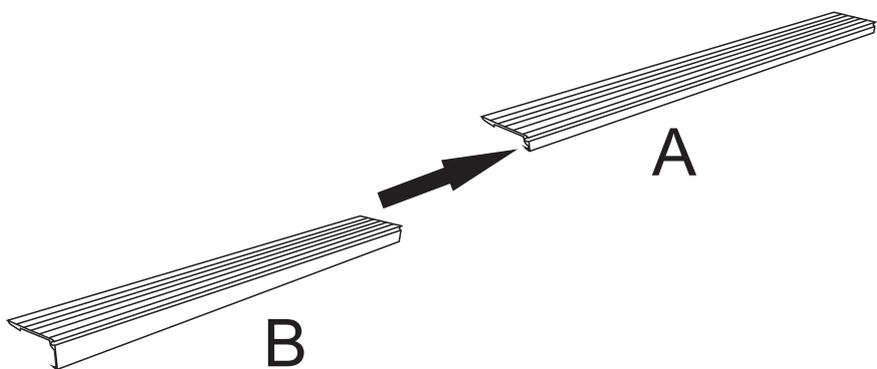
713-086

IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY

ASSEMBLY & INSTRUCTION MANUAL

 A x 1	 B x 1	 C x 1	 D x 1
 E x 1	 F x 1	 H x 2	 I x 5 +1(spare)

ALUMINUM THRESHOLD RAMP INSTRUCTION MANUAL

MAXIMUM WEIGHT CAPACITY IS 700LBS.

(Weight rating is base on evenly distributed load on 2 axles, 350LBS capacity on a single axle.)

1. The aluminum transitions entry ramp creates an anti-slip, sturdy and seam-free surface.
2. It is designed for loading wheelchair, scooter, walker when going through doorways and raised landings throughout the home.
3. Easily add or remove the interlocking sections to build the suitable threshold height.
4. The ramp is lightweight and can be easily folded and transported.

WARNING

1. Make sure all the ramp sections are locked well and fixed with the supplied screws, the top attaching lip is secured onto the step or landing before use. We strongly recommend using the supplied safety pins to prevent ramp kick out.
2. NEVER exceed the rated capacity of your ramp.
3. ONLY use ramp with a qualified helper, and always use your lap belt.
4. Use caution at all times when loading and unloading.
5. ONLY use ramp on a stable and level surface.
6. Ramp surfaces will have less traction when wet. Use caution.
7. The ramp is not a toy. DO NOT use ramp other than its designed purpose.
8. ALWAYS follow manufacturer recommendations for wheelchair or scooter.

MAINTENANCE

1. It is user's responsibility to check the ramp and its components before each use. If the ramp is damaged, it should NOT be used.
2. Maintain the ramp by regular cleaning.

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